


GETTING STARTED with SaniCube

- Remove Bottle from SaniCube.
- Remove Charging Cable from Bottle.
- Fill Bottle with Recommended Liquid and screw bottle into SaniCube.
- Turn Master Power On, and remove Sticker from Cube Bottom.
- Place SaniCube on a surface and wait 6 seconds boot-up.
- Wave your hand over SaniCube until the atomizer delivers your first spray - and clean your hands.

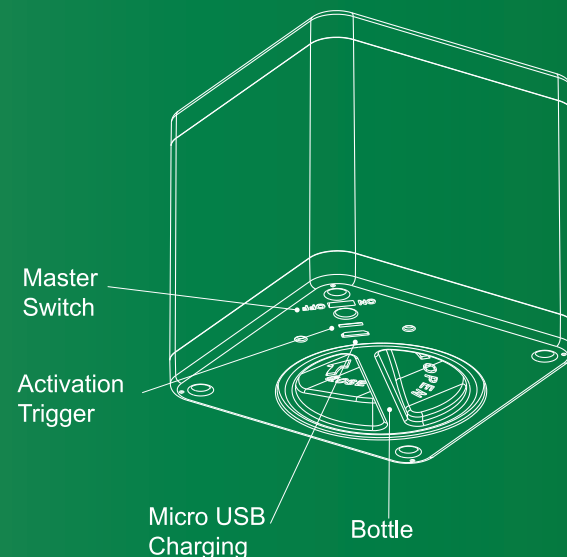

RECOMMENDED LIQUID

You can use any disinfectant in SaniCube as long as it has the same or less viscosity as water.

- **YOU CAN USE:**
 - 200ppm HypoChlorous Acid (HOCl, or HClO)
 - Any liquid alcohol hand sanitizer (if using alcohol, you must use our ice blue color SaniCubes only).
- **DO NOT USE:** A disinfectant made from dissolved effervescent tablets - these liquids actually have solids in suspension which could cause the atomizer to prematurely fail. Using anything other than what is listed above will void the warranty of this device.



SaniCube is a prestige, high quality, low volume, drip-free motion-sensor hand-cleaning unit for home and commercial applications. This means 'not touching' where other people 'touched' in order to clean your hands.

Whether a child, a parent, a customer or staff member, if anyone waves their hand over SaniCube, the atomizer will apply a short burst vapor of disinfectant to clean their hands, phone, keys, money, etc.. and there is no need to dry with a tissue.

SaniCube is able to be located wirelessly like on a table top, or connected to power via the CHARGE STATION with built-in USB cable.

CHILD SAFETY FEATURE

Whether for a curious adult or child, we designed SaniCube to de-activate the atomizer if the cube is picked up off the surface to ensure they cannot look straight at the motion sensor and get sprayed in their eyes.

We also adjusted the range of the motion sensor so it is mostly sensitive at 10-15cm from the sensor.

SaniCube will also de-activate on the CHARGE STATION (if applicable).

CHARGING THE BATTERY

To charge Sanicube, use the single or 4-station charging base provided.

DO NOT CHARGE WITH ANY OTHER CHARGING DEVICE.

Simply set SaniCube in charging base.

Sanicube will operate while in charging base providing continuous use.

DIRECT SUNLIGHT

The motion sensor in SaniCube is a premium infra-red sensor meaning it can be used in direct light on the inside of a window.

As pure sunlight also includes the infra-red spectrum, SaniCube will not work in direct sunlight (for example on an outdoor table).

CLEANING YOUR CUBE

Wipe down your CUBE with warm soapy water, and wipe dry with a microfiber cloth.

DISCLAIMER

SaniCube is a Disinfectant Delivery Device. The manufacture takes no responsibility for liquids used in SaniCube.

If you are using alcohol (in certain models), remember it is flammable, so do not use SaniCube with alcohol near an open flame. 200ppm hypochlorous acid (HOCl) is recommended as it's non-flammable and non-toxic.

WARRANTY

SaniCube is sold and warranted for one year from date of purchase. Improper care and use of unapproved liquids will void warranty.

CLEANING YOUR HANDS

Just wave your hand about 4-6" (10-15cm) over SaniCube to activate the motion sensor and a two-second dose of disinfectant vapor will spray onto your hands.

Rub your hands together, remembering your finger tips and thumbs and the disinfectant will slowly evaporate and leave your hands clean.

It is not needed to use a paper towel - your hands will be dry in 10-15 seconds.

CHARGING INDICATORS

SaniCube has a battery power indicator built into the rectangular motion sensor panel in the top of SaniCube.

When you activate SaniCube, the sensor will emit a blue light for 3 seconds.

Sensor
3 SECOND LIGHT:
Battery has operational power

3 x 1 SECOND FLASHING LIGHT:
Battery is low power

Charging Indicator (on top corner)
RED LIGHT: Charging BLUE LIGHT: Charged

LOW-VOLUME USAGE

1 x 180 ml BOTTLE will deliver over 500 x 2 second sprays.

This is just 0.35ml per application. We limit each dose to 2 seconds so we are only applying your hands exactly the amount of disinfectant you need. No drips and no need for tissues to dry your hands. Conveniently Efficient!



MICROBE NINJA™

microbeninja.com/sanicube

POWERING ON / OFF

MASTER SWITCH: If you are ever not using SaniCube, there is a micro-switch underneath that isolates the internal power so there is no battery usage.

CHILD SAFETY FEATURE: SaniCube has a Child Safety Switch which deactivates SaniCube if it were ever picked up off a surface, the motion sensor is de-activated.

REMOVING AND REPLACING BOTTLE

SaniCube's unique 180ml bottle is available with a screw cap. This means you can have spare bottles to refill, or you can buy 'ready-to-load' bottles from your supplier.

To remove the bottle, hold SaniCube upright (operational orientation) and turn the grip in the base of the bottle anti-clockwise. Don't worry, we designed it to have a really good grip.

To replace the bottle, hold SaniCube upright, and screw the bottle clockwise. You will notice it is not so easy to grip - this is to ensure that you cannot over-tighten the bottle and damage the o-ring inside.

CHANGING THE ATOMIZER

The ATOMIZER device will deliver up to 1,000,000 x 2 second sprays. In the event of premature failures, the atomizer is able to be replaced quite simply.

To change the ATOMIZER, open the 4 bolts in the base of the CUBE - lift the top of SaniCube off the housing. Pick the Atomizer disc up from its housing so you can identify where the cable clips into the circuit board.

Unplug the Atomizer cable plug. Take your replacement Atomizer disc, plug in the cable to the circuit board, and place the Atomizer Disc centered in the top of the circuit board. Reassemble your SaniCube.

CHARGING SaniCube

To know when your SaniCube needs recharging, it will either a) stop atomizing or b) the light in the CUBE BASE (if applicable) will stop glowing.

USB CABLE: Turn SaniCube on its side and plug the USB CABLE (supplied) to the MICRO-USB port in the bottom of SaniCube. Connect the USB plug to any 1 AMP power adapter (this is any universal appliance adapter).

SINGLE CHARGE STATION: Place SaniCube on the CHARGE STATION - it only fits with one orientation. Connect the USB plug to any 1 AMP power adapter (available separately).

REFILLING THE BOTTLE

SaniCube bottles are available as an accessory so in a commercial application, you can have spare bottles filled in the storeroom, and then refilling bottles can be scheduled as a task, and bottles are always ready for replacing in the SaniCubes.

Refilling the bottle is simple - pour your disinfectant up to the bottom of the cap thread and then screw down the cap to seal.

CHANGING THE COTTON STEM

The Cotton Stem is what feeds the disinfectant to the Atomizer. If your Atomizer spray looks weak, you can rotate or replace the cotton stem. To change the Cotton Stem, remove the bottle from the underside of SaniCube. Place it somewhere safe so it cannot be knocked over.

Take hold of the Cotton Stem housing and turn it counter-clockwise. It will come away very easily. Replace it with one of the two spare Cotton Stems supplied with SaniCube. Return the Cotton Stem Housing with a clockwise turn to refit it, and refill the bottle.